Duocité B - Allée de la Mandallaz
74370 Metz-Tessy
France
Office: +33 (0)4 50 05 39 43
contact@b-ad.fr

Date: 23/10/2024

Operator: VA

ID Client: Cooper

B-ad Number: VE11447

B-ad Version: V9

Product Name: Eroxon IFU Nordics

Software: IND

Size: 300 x 416 mm

AW reference: 207391 LC26

Pharmacode: LC22 (0111)

Number of Colors: 1

Countries:

NO
DK
FI
SE

Languages:

EN
NO
DK
FI
SE
EE
LV
LT

Process Colors:

Black

PMS:

Tech. description:

Signature:

Date:

Last Name, First Name:

EN

These instructions for use contain important information for you. Please read it carefully before using Eroxon®.

WHAT IS IN THESE INSTRUCTIONS FOR USE

1. What **Eroxon®** is and what it is used for
2. What you need to know before using **Eroxon®**
3. How to use **Eroxon®**
4. Possible side effects
5. How to store **Eroxon®**
6. Contents of the pack and other information

1. WHAT EROXON® IS AND WHAT IT IS USED FOR

Eroxon® is a clinically proven gel treatment for adult men (aged 18 years and over) with **erectile dysfunction (also known as impotence)**. This is when a man cannot get or keep an erection hard enough for satisfactory sexual activity.

Eroxon® stimulates blood flow in the penis through a unique physical cooling then warming effect, helping you get and keep an erection hard enough for sex.

Eroxon® will only help you to get an erection if you are sexually excited (stimulated).

2. WHAT YOU NEED TO KNOW BEFORE USING EROXON®

Do not use Eroxon® if:

- You are **allergic** to any of the ingredients (listed in section 6).
- You have been advised by your doctor to **avoid sexual activity**.
- You have any disease or **deformity of your penis** (such as Peyronie’s Disease).
- The skin on your penis is **red or sore**, or appears **damaged or broken**.

Warnings and precautions:

Sexual activity can put extra strain on your heart. If you have a heart problem or think you might have a heart problem you should speak to your doctor.

It is important to have a check-up with your doctor as soon as possible within the first 6 months of using Eroxon® to ensure that your erection problems are not caused by another serious health condition.

Even if you feel healthy, your erection problems may be linked to other serious health conditions. Erectile dysfunction may be an early sign of:

- heart and blood vessel problems
- diabetes
- high blood pressure
- kidney disease
- multiple sclerosis
- high cholesterol
- liver disease or alcoholism

You should discuss your symptoms and these conditions with your doctor if you have not already done so.

Before using **Eroxon®** in order to ensure you are fit enough to have sex, tell your doctor if you:

- Get very breathless or feel any pain in the chest with light or moderate activity (e.g. walking briskly for 20 minutes or climbing two flights of stairs). Your doctor may need to check whether your heart can take the additional strain of having sex.
 - Have ever had an erection that lasted more than 4 hours even without any physical or psychological stimulation (called priapism) or have any condition that can cause priapism. Conditions which can cause priapism include sickle cell anaemia (an abnormality of red blood cells), leukaemia (cancer of blood cells), or multiple myeloma (cancer of bone marrow).
- Other important information:**
- **Eroxon®** will only benefit you if you have problems getting or maintaining an erection.
 - Erectile Dysfunction treatments may not work for everyone. Talk to your doctor about alternative treatment options if **Eroxon®** does not work for you.
 - For most men, **Eroxon®** will work on their first or second attempt. If it has been sometime since you were able to get or keep an erection, it may take a few attempts before you achieve the desired effect.
 - **Eroxon® is not a contraceptive** and contains **no spermicidal ingredients**. It does **not** protect against sexually transmitted infections (STIs).
 - **Eroxon® is only compatible with latex condoms**.
 - **Eroxon® is not a lubricant**. However it is safe to use **Eroxon®** with personal lubricants.
 - You can **improve** your erectile function by making lifestyle changes; these include exercising regularly, reducing stress, giving up smoking and avoiding excessive alcohol use. If you are feeling depressed or anxious, talk to your doctor or pharmacist, who will be able to provide you with further advice.

Any serious incident that has occurred in relation to this device should be reported to the manufacturer and the Competent Authority of the country in which you are located.

3. HOW TO USE EROXON

Eroxon® is supplied in a single dose tube.

When to use: Apply the gel just before sex.

How to open the tube: Unscrew the cap. Pierce the tube seal with the top of the cap. Squeeze tube firmly to empty all the contents which will provide the correct dose (approximately a pea-sized amount of gel).

How much to apply: Apply the contents of one tube to the **head of your penis (a)**. If you are uncircumcised, gently pull your foreskin back as far as comfortable before applying the gel.

How to apply: **Massage the gel onto the head of your penis for about 15 seconds.** You can apply the gel yourself or your partner can apply the gel as part of foreplay.

Eroxon® will only help you get an erection if you are sexually aroused. You and your partner should engage in foreplay just as you would if you were not using a treatment for erectile dysfunction.

4. POSSIBLE SIDE EFFECTS

Rarely, using **Eroxon®** can cause skin irritation (feelings of burning, itching or soreness). Stop using **Eroxon®** if you or your partner experience any irritation. Speak to your doctor or pharmacist if the irritation persists.

5. HOW TO STORE EROXON®

Check the expiry date on the tube before use. The expiry date refers to the last day of that month. If the tube is damaged, do not use it. Throw the tube away and use a new one.

6. CONTENTS OF THE PACK AND OTHER INFORMATION

Eroxon® is available in packs of 4 single-dose tubes.

Ingredients: Aqua, ethanol (35%), propylene glycol, glycerine, carbomer, potassium hydroxide.

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Futura Medical Developments Ltd. Guildford, GU2 7